PMMA for brain®

# Gebruiksaanweisung

Instructions for Use

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# PMMA for brain®

<b>EN</b>	Gebrauchsanweisung PMMA for brain®.....	01-05
<b>DE</b>	Gebrauchsanweisung PMMA for brain®.....	06-10
<b>FR</b>	Notice d'utilisation PMMA for brain®.....	11-15
<b>IT</b>	PMMA for brain® - Istruzioni per l'uso .....	16-20
<b>ES</b>	Instrucciones de uso de PMMA for brain®.....	21-25

# PMMA for brain® Instructions for Use

EN

## Product description:

PMMA for brain disks are made of polymethylmethacrylate. They are used to produce resin crowns, bridges and denture frameworks for provisional rehabilitation in the anterior and posterior regions.

Framework material	Polymethylmethacrylat
Temporary cementation	Yes
Definitive cementation	No

Objects are individually fabricated to your digital design specifications, such as anatomic contour, framework and wall thickness, connector diameter and cementing gap, using Cercon® brain expert und brain® Xpert. Please always use the design software in its current version, since this also affects the production of your objects.

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- Crowns
- Multi-unit bridges (with two pontics between abutment crowns in the anterior region – up to the second molar – and one pontic between abutment crowns in the posterior region). The restorations should not be worn for longer than twelve months.

## Contraindications:

- This medical product may not be used in patient hypersensitivity to polymethylmethacrylate or one of the other ingredients.
- Cantilevered-pontic bridges are fabricated at the ordering laboratory's risk.

## Warnings:

Possible cross-reactions or interactions of this medical product with other medical products or material already present in the oral environment must be taken into consideration by the dentist when selecting this medical product.

- Processing PMMA for brain disk will result in the release of dust that may cause irritation of the eyes, skin or respiratory passages. The legal threshold limit value (TLV) for methylmethacrylate in Germany is 210 mg/m<sup>3</sup>. To ensure safe and sustainable compliance, the measured value must not exceed 25% of the TLV. This in turn required that the room in which the Cercon® brain expert/brain Xpert is used to process this material (max. 2 disks per 8 h) must have a volume of at least 28 m<sup>3</sup>. In rooms with a smaller volume, the air exchange rate must be increased by a suitable HVAC installation.

**Precautions:**

Please note:

- Keep product dust away from eyes.
  - Avoid any contact with mucosa.
  - After use, wash your hands and apply a hand cream.
  - Do not smoke, eat or drink while handling the product.
  - Do not swallow the product.
  - Do not inhale dust particles during grinding.
  - Use local vacuum suction and suitable mouth/face protection during grinding.
- Wear gloves when changing or disposing of the dust filter.

The safety and warning notes listed here describe how to use our system components in a safe and risk-free manner. Notify the dentist in charge of all factors described above if you use this medical product for a custom design and make sure to comply with the applicable Material Safety Data Sheets (MSDS).

**Adverse effects:**

If properly processed and used, adverse effects of this medical product are highly unlikely. However, reactions of the immune system (such as allergies) to substances contained in the material or localized paraesthesia (such as taste disturbances or irritation of the oral mucosa) cannot be completely ruled out as a matter of principle. Should you hear or receive information about any adverse effects – even when doubtful – we would like to request notification.

## Processing instructions:

<b>Framework dimensions for the anterior and posterior regions:</b>	
Wall thickness, single crowns	1.0 mm
Margin thickness, single crowns	0.2 mm
Wall thickness, bridges	1.0 mm
Margin thickness, bridges	0.2 mm

<b>Additional dimensional requirements for the anterior region:</b>	
Number of pontics	2 to the second premolar
Connector cross-section	Minimum 12 mm <sup>2</sup>

<b>Additional dimensional requirements for the posterior region:</b>	
Number of pontics	1
Connector cross-section	Minimum 12 mm <sup>2</sup>
Cantilever pontic at tooth position (only one pontic, up to one premolar width)	Contraindicated
Connector cross-section for this cantilever pontic	-

### Technical specifications:

- Density: 1.19 g/cm<sup>3</sup>
- Solubility: 0.2 µg/mm<sup>3</sup>
- Flexural strength: > 90 MPa
- Water absorption: 26.5 µg/mm<sup>3</sup>
- Flexural modulus: 2773 MPa

### Composition (in mass%):

- Polymethylmethacrylate and cross-linked copolymers of methacrylic acid > 99%.
- Pigments (predominantly iron oxide pigments) and benzoyl peroxide < 1%.
- Polymethylmethacrylate may contain up to approximately 1% residual monomer.
- Contains methylmethacrylate, dibenzoyl peroxide.  
May cause allergic reactions.

### Manual processing:

To avoid imprecisions of fit, avoid excessive heat during finishing and polishing.

**Repair and customization:**

PMMA for brain can be repaired or customized with the following products:

- PMMA for brain Dentin/Enamel: Tooth-coloured PMMA plastic
- PMMA for brain Dentin Liquid
- PMMA for brain Bonder: Adhesive

**Contraindications/interactions:**

In patient hypersensitivity to one of the ingredients, these products may not be used at all or only under the particular scrutiny of the dentist or physician in charge. The composition is available on request if needed. Known cross-reactions or interactions of this medical product with other materials already present in the oral environment must be taken into consideration by the dentist in charge when using this product.

The product properties are based on adherence to and compliance with these Instructions for Use.

**Processing notes:****PMMA for brain Dentin/Enamel****Dosage:**

Mixing ratio PMMA for brain Dentin/Enamel : Liquid, 1 g : 0,7 ml

Mixing time 30 s\*

Swelling time 15 s to max. 45 s following the mixing time\*

Flow phase 240 s

Polymerization 15 min. in a pressure pot at 45°C and 2 - 2.5 bar pressure

Wherever possible, use class IV dental stone for cast fabrication. Always wet the cast thoroughly and isolate the plaster from the using a commercially available alginate.

**Bonding:**

- Degrease and abrade the contact surfaces before processing.
- Avoid any contamination by wax or isolating compounds.
- In teeth with cross-linked resins, it is recommended to first apply PMMA for brain Bonder.

\*All values assume a room temperature of 21°C.

**PMMA for brain Bonder:**

- Remove residual wax, grease, grinding dust and other contaminants before applying. After steam-cleaning ensure that all parts, including undercuts, are thoroughly dry. When drying with compressed air, make sure that no contamination occurs (e.g. by oil residue from the compressed-air system).
- Roughening the surfaces to be joined. For larger surfaces, it may be advisable to provide small retention grooves.
- Wet the surface to be bonded with PMMA for brain Bonder and apply enough bonder for the contact surface to stay moist for 5 min. If the contact surface has dried of all PMMA for brain Bonder during this time, re-moisten with PMMA for brain Bonder. The exposure time must not be less than 5 min. PMMA for brain Dentin/Enamel must be applied no later than 10 minutes after conditioning the surface.
- Avoid any contact with plaster or with materials containing silicone.

**Finishing and polishing:**

- To avoid inaccuracies of fit after polymerization, avoid excessive heat during finishing and polishing.

**Special notes:**

- Close containers tightly after use.
- Do not use the material after the stated expiration date.
- Store in a dry and cool place (up to 25°C). Avoid direct sunlight.
- Indicate the batch (LOT) number for every transaction that requires an identification of the material.
- Notify the dentist or physician in charge of all factors described above if you use this medical product for a custom construction. When working with these materials, make sure to comply with the product-specific Material Safety Data Sheets (MSDS).

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DeguDent GmbH