**DENTSPLY**  
TRUBYTE

# Luci-Sof®

Injectable Soft Denture Liner

Injectable Soft Denture Liner

Ribasatura morbida iniettabile per protesi

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**MODE D'EMPLOI**

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## Indications for use:

The Luci-Sof® Injectable Soft Denture Liner is intended for use as a long-term resilient lining material for removable dentures. This product requires the use of the Success Injection System.

The Luci-Sof Injectable Soft Denture Liner consists of two components:

1. **Luci-Sof Injectable Soft Denture Liner Material:** a heat polymerizable, one-component silicone putty that is based on dimethylpolysiloxane, dimethyl, methyl hydrogen siloxane copolymer; reinforced with amorphous silica. Benzoyl peroxide is the initiator for polymerization.
2. **Luci-Sof Bonder:** a heat polymerizable methacrylate monomer liquid and methacrylate polymer blend.

## Contraindications:

1. Luci-Sof Bonder is contraindicated for patients and users with history of allergic reactions to methyl methacrylate monomer. Do not use Luci-Sof Injectable Soft Denture Liner on dentures that will result in the denture base acrylic being less than 2 mm in thickness.

## Warnings:

1. Luci-Sof Bonder contains polymerizable monomers, which may cause skin sensitization (allergic contact dermatitis) and/or other allergic reactions in susceptible individuals. Wash area thoroughly with soap and water after contact. If skin sensitization or other allergic reaction occurs, discontinue use. If dermatitis or other symptoms persist, promptly seek medical advice.
2. Avoid inhalation or ingestion of Luci-Sof Bonder. High vapor concentration can induce headache, nausea, and irritation of eyes and respiratory system. Contact with eyes may cause possible corneal damage. Excessive long-term exposure may be associated with other, more serious health effects. Monitor air quality per OSHA standards.

**Inhalation:** Move subject to fresh air. Give oxygen or artificial respiration as required.

**Ingestion:** Contact your regional Poison Control Center immediately.

**Eye Contact:** Flush eyes promptly with copious amounts of water for 15 minutes and consult a physician. Wash skin with soap and water.

## Precautions:

1. To achieve a complete cure in 60 minutes, a "rolling" boil must be maintained through the entire cure cycle. If the water bath temperature drops below boiling at any time, the cure time must be extended.
2. Do not use Luci-Sof Injectable Soft Denture Liner on dentures that will result in the denture base acrylic being less than 2 mm in thickness.
3. Users with special skin problems, cuts or abrasions should wear protective gloves.
4. When grinding prosthodontic resins, proper ventilation, masks and vacuum systems should be used.
5. Refrigeration of Luci-Sof Injectable Soft Denture Liner is recommended. Always bring to room temperature before use. Do not store next to amine containing chemicals (i.e. acrylic repair liquid, adhesive accelerator, etc.) as this may cause discoloration of the Luci-Sof material.
6. Store Luci-Sof Bonder below 80° F (27° C) and away from direct sunlight. May be refrigerated to extend shelf life. Always bring to room temperature before use.
7. Eugenol-containing dental materials should not be used in conjunction with this product since they may interfere with hardening and bonding.
8. Unused Luci-Sof Injectable Soft Denture Liner should be polymerized prior to disposal.
9. Luci-Sof Bonder is a hazardous material. Dispose of it in accordance with Federal State and Local regulations.

## Adverse Reactions:

1. Corneal damage, headache, nausea, vomiting may occur with exposure to methyl methacrylate monomer. (See Warnings and Precautions statements.)
2. Allergic contact dermatitis and/or other allergic reactions may occur in susceptible individuals.
3. Particulates will be generated when grinding prosthodontic materials. Eye, skin and respiratory irritation may occur if appropriate engineering controls are not used.

## Step-by-Step Instructions for Relining a Denture

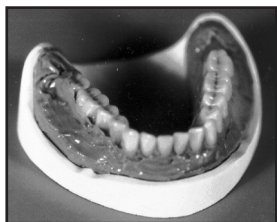
Fig. #A

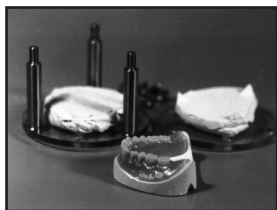
Fig. #B

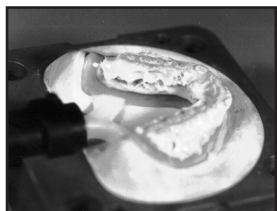
Fig. #C


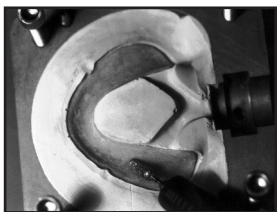
Fig. #D

Luci-Sof Injectable Soft Denture Liner is a long-term silicone, heat-cured soft denture liner material. This product requires the use of the Success Injection System.

1. Pour a stone cast of the impression in the denture to achieve the desired contour (**see Fig. #A**).
2. Trim excess impression material smooth with wax, and finish to final shape around the periphery. This preparation will help to minimize finishing. Optional method is to use a reline jig method before investing. This provides easy access for denture preparation and will help to minimize finishing (**see Fig. #B**).
3. Invest in bottom half of the flask.
4. Sprue according to Success System directions (Success® Sprue Wax, Item #904584, is recommended). For lower dentures, place sprues as far back on the lingual surface as possible (**see Fig. #C**).  
**Optional:** Use Dentsulate® insulating putty to aid in deflasking.
5. Invest top half of the flask.
6. Open the flask and remove wax with boiling water.
7. Remove the impression material.
8. Wash with detergent and water, and rinse with clean boiling water.
9. Grind the denture surface with a clean Faskut® Carbide Cutter or coarse diamond stone to remove tray adhesive and sufficient acrylic. Grind flange border to a 90° butt joint.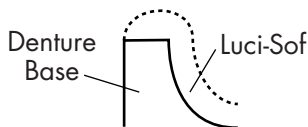

Avoid knife edges. A rough and clean surface is necessary to achieve a superior bond to the denture base. (**see Fig. #D**)

Fig. #E

10. Shake the Luci-Sof Bonder well and generously coat the areas of the denture to be bonded. (see Fig. #E)
11. **Leave the flask open and allow the Luci-Sof Bonder to dry for 30 minutes to achieve a superior bond of the soft liner to the denture base.**

12. Close the flask and prepare it for injection. (see Fig. #F)

13. Inject the Luci-Sof Injectable Soft Denture Liner material for 5 minutes. The injection cylinder should remain down for the full 5 minutes (see Fig. #G).

14. Remove the flask from the injection unit and install the pressing device.

15. Immediately process the flask for 60 minutes in boiling water (see precaution #1). Cool 15 minutes on the bench and then place in room temperature water for 15 minutes (see Fig. #H).

16. Remove excess material with a scalpel, contour with a Faskut® Luci-Sof Bur or your favorite silicone adjustment instruments (see Figs. #I and #J).

17. Polish with Tripoli and felt wheel at high speed (15,000 rpm) (see Fig. #K).

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**DENTSPLY International Inc.**
**DeguDent GmbH**